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Ziegelei 1
D-72336 Balingen
E-Mail: info@kern-sohn.com

Tel: +49-[0]7433- 9933-0
Fax: +49-[0]7433-9933-149
Internet: www.kern-sohn.com


KERN MSC

Type MSC 100N
Version 1.3
2017-03

D	2
GB	9
F	16
BG	23
CZ	30
DK	37
E	44
EST	51
FIN	58
GR	65
H	72
HR	79
I	86
LT	93
LV	100
NL	107
NO	114
P	121
PL	128
RO	135
SE	142
SK	149
SLO	156

MSC-BA-EU-1713

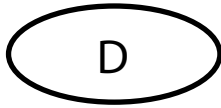KERN MSC

Version 1.3 2017-03

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KERN MSC

Version 1.3 2017-03

Operating instructions Mechanical height measuring rod

1 Technical Data

KERN (Type)	MSC 100N
Trademark	MSC 100
Measuring range	7.5 – 100 cm
Readability	1 mm
Tolerance	5 mm
Dimensions (D x H x W) mm	1040 x 55 x 290 mm
Temperature range	+10°C to +40°C
Storage temperature	- 20°C ... + 60°C
Net weight	approx. 800 g
Admitted as medical product as per Guideline 93/42/EWG (CE 0297)	Category I with measuring function

2 Declaration of conformity

To view the current EC/EU Declaration of Conformity go to:

www.kern-sohn.com/ce

3 Explanation of the graphic symbols

This marking shows that this weighing balance is in conformity with EU Directive 93/42/EEC and inside the European Community is classified as medical device.

SN WY 170563

Designation of the serial number of every device, applied at the device and on the packaging.
(Number as an example)

2017-03

Designation of the serial number of every device, applied at the device and on the packaging.

Number here as example

Identification of the manufacturing date of the medical product.

Year and month here as example

Kern & Sohn GmbH
D-72336 Balingen, Germany
www.kern-sohn.com

„Attention observe accompanying document“, or „Follow operating instructions“

Temperature limit indicating the lower limit (-20°C) and the upper (+60°C) limit
(storage temperature on packaging)

10°C / 40°C

Temperature limitation in the application indicating the lower and the upper limit
(Allowable ambient temperature)

Temperature indication as example

4 Basic Information (General)

This height measuring rod according to guideline 93/42/EWG is provided for determination the body height in medical practice and for medical survey, examination and treatment.

Defined purpose

Indication:

- Determining the body height in the medical practice area.
- At an upright standing person, or at a lying baby, the height measuring rod is applied to head and feet in a manner that on the display or scale the body height value can be read.

Contraindication:

No contraindication known.

Proper use

This height measuring rod is used to determine the height of persons while standing upright, or of lying babies, according to its model, in medical treatment localities. This height measuring rod is suitable to detect, prevent and survey diseases.

Ensure that the height measuring rod may only come into contact only with uninjured skin.

Improper Use

The structure of the height measuring rod may not be modified. This may result in incorrect measuring results, safety-related defects as well as destruction.

The height measuring rod may only be used according to the described conditions. Other areas of use must be released by KERN in writing.

Warranty

Warranty claims shall be voided in case

- Our conditions in the operation manual are ignored
- The appliance is used outside the described uses
- The appliance is modified or opened
- Mechanical damage or damage by media, liquids, natural wear and tear.
- Non correct implantation or assembly.

Test device control

For height measuring rods, we recommend a metrological examination of the accuracy of the measuring rod, however, this is not mandatory as the determination of human body height involves rather large, intrinsic inaccuracies.

5 Basic Safety Precautions

- ⇒ Carefully read this operation manual before setup and commissioning, even if you are already familiar with KERN balances.
- ⇒ All language versions contain a non-binding translation.
The original German is binding.

Staff training

The medical staff must apply and follow the operating instructions for proper use and care of the product.

Avoiding contamination

In order to avoid crossed contamination (fungal disease,...) the parts coming into contact with the patient must be cleared regularly.

Recommendation:

After every application which could have a potential contamination as consequence (e.g. direct contact with the skin).

Safety Instructions

- Observe operating instructions.
- Please use no other than the supplied fixtures for mounting.
- During assembly ensure that the height measuring rod is correctly assembled, (see chap. 6).
- When moving the height measuring rod ensure that the person to be measured can not be injured
- If a measuring flap exists, ensure that this will be folded-in after every use as otherwise there exists risk of injury.
- To prevent risk of injury, ensure that the measuring flap is returned to folded-in position after each use.
- The body height measuring emits only reliable values, when the back and the head are straightly aligned, nevertheless the body height of a person may change about some cm during a day.

6 Unpacking/installing

Testing upon acceptance





Please check packaging as well as the device on receipt for possibly visible external damages.

Packaging/Return

Keep all parts of the original packaging for a possibly required return.
Only use original packaging for returning.


7 Body height measurement

⇒ Open measuring flap, as shown on illustration.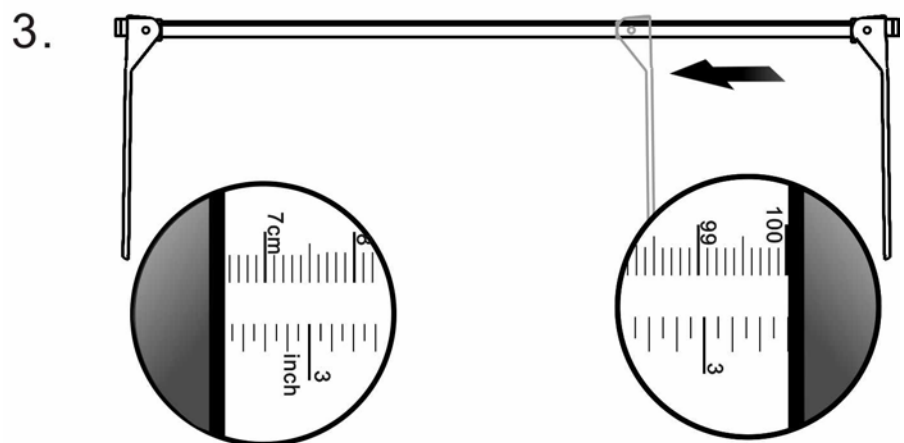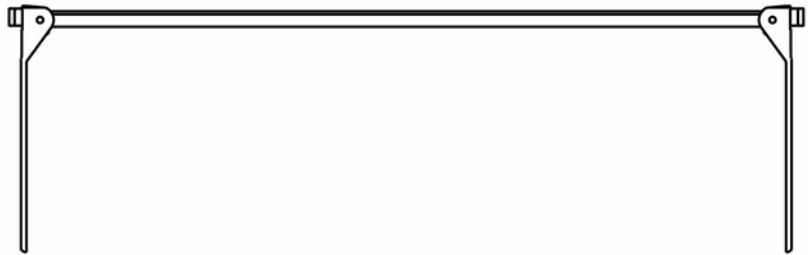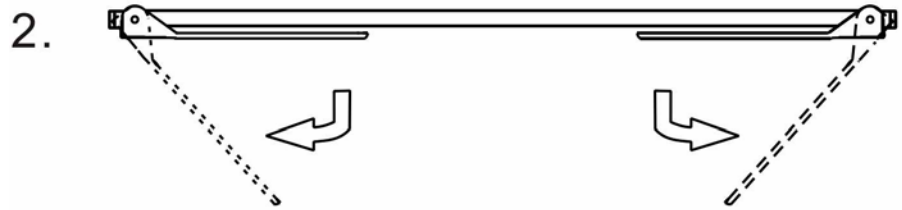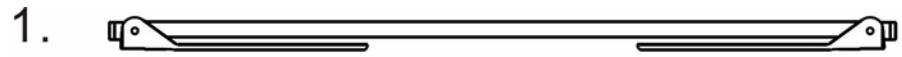


Fixed stop

Movable stop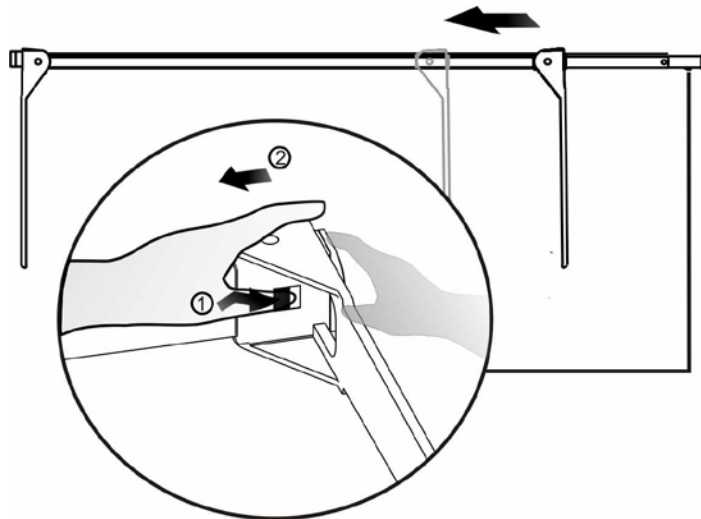

⇒ To move the stop, loosen the clamping by pressing the black button [1].

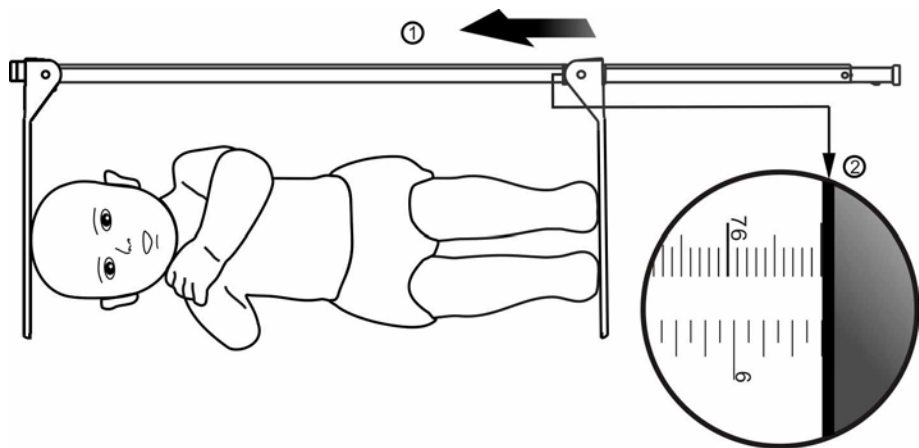- ⇒ Position the baby in a way that the head touches gently the fixed stop.
- ⇒ Shift caliper gauge acc. to fig. till the heel stop.
- ⇒ Read body height on red marking.

8 Cleaning, Maintenance, Disposal

Cleaning Disinfection	<p>The body height scale should be cleaned with household detergents or commercially available disinfectants, e.g. 70% isopropanol. We recommend a disinfectant suitable for wiping disinfection. Please follow manufacturer's instructions.</p> <p>Do not use any polishing or aggressive cleaning agents (white spirit, petrol etc.) as they may damage the high-quality surface.</p>
Sterilisation	<p>Sterilisation of the appliance not allowed.</p>
Service, maintenance	<p>The appliance may only be opened by trained service technicians who are authorized by KERN.</p>
Disposal	<p>Disposal of packaging and appliance must be carried out by operator according to valid national or regional law of the location where the appliance is used.</p>

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